

# Operating Instructions for Electronic Wheelchair Scale




**Caution!** Load Cell damage can occur if scale is allowed to fall from storage position.

- ① From transport position, lay scale on floor.

**DO NOT** remove the Storage Pin or operate Column Release Lever prior to this operation.

**DO NOT USE SCALE ON CARPETED FLOORS!**

- ② Remove the Storage Pin.

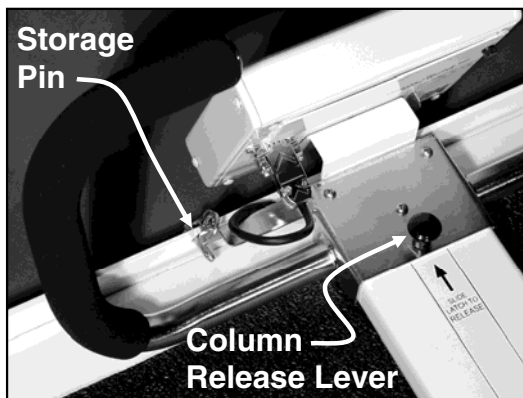

- ③ Slide the Column Release Lever to release column and lift to full up-right and locked position.

 **DETECTO.**  
[www.Detecto.com](http://www.Detecto.com)



# Operating Instructions for Electronic Wheelchair Scale

- ④ Press **ON** key located on front of indicator.
- ⑤ When indicator reads Zero, press **TARE** key, enter wheelchair weight, then press **ENTER** key. The indicator will show a negative weight equal to weight of wheelchair.



- ⑥ Position patient on scale by pulling wheelchair onto scale. **NEVER PUSH** patient on or off scale. Lock wheelchair wheels.
- ⑦ When weight reading is steady, read the patient's weight.

- ⑧ Remove patient from scale by backing down the ramp.



**NEVER** leave a patient unattended while on the scale. Failure to maintain control of patient and/or wheelchair at all times can result in serious injury to patient.

- ⑨ For storage, use column release to release column and fold down to locked position. Replace storage pin (see *warning label on scale*). Store in an upright position, near a wall for maximum protection against damage.

