
Head Switch #705

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Ideal for wheelchair patients!

The head switch is activated with very light pressure and can also be activated by a knee, elbow, etc. Attaches easily to many of our switch mounting systems, (sold separately #1550 or #1557). Includes our Heavy Duty Ball Joint #1010 with a threaded male 1/4-20 mounting post.

Operation:

1. The Head Switch does **not** require any batteries to operate.
2. The Head Switch comes with a removable Heavy Duty Ball Joint. To remove the Heavy Duty Ball Joint unscrew the Head switch from the top of the Heavy Duty Ball Joint.
3. Plug the Head Switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Head Switch and your toy/device is plugged in all the way. There should be no gaps.
6. Simply press anywhere on the Head Switch's top to activate your toy/device.
7. The toy/device will only remain activated while the Head Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Head Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Head Switch and your toy/device is plugged in all the way. There should be **no** gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Head Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.