

# Legacy Harness

## **NEW** BANDOLIER HARNESS

**Features:**

- Features elastic loops for adjustment of the strap intersection
- Extended upper straps go over the shoulder
- Quick release buckles located near the chair back
- Stand behind user and pull finger loops to adjust top straps
- Hardware to attach to chair included



| Description | Part Number | Padded Area Dimensions |
|-------------|-------------|------------------------|
| Small       | 30639       | 1.5" x 14"             |
| Medium      | 30640       | 1.5" x 16"             |
| Large       | 30641       | 1.5" x 18"             |

| Bandolier Harness | Top Strap sections adjustment w/hardware | Bottom Strap adjustment w/hardware |
|-------------------|------------------------------------------|------------------------------------|
| Small             | 13" maximum                              | 15" maximum                        |
| Medium            | 13" maximum                              | 15" maximum                        |
| Large             | 13" maximum                              | 15" maximum                        |



## PADDED X-STYLE HARNESS


**Features:**

- Cushioned harness with reinforced webbing
- X-Style harness has minimal stretch
- Extended upper straps go over the shoulder
- Quick release buckles located near the chair back
- Stand behind user and pull finger loops to adjust top straps
- Hardware to attach to chair included



| Description | Part Number | Padded Area Dimensions |
|-------------|-------------|------------------------|
| Small       | 30519       | 1.5" x 10"             |
| Medium      | 30520       | 2" x 13"               |
| Large       | 30521       | 2.5" x 16"             |

| X-Harness | Top Straps | Finger Loop Straps, without hardware | Bottom Straps, without hardware |
|-----------|------------|--------------------------------------|---------------------------------|
| Small     | 8" long    | 10" long                             | 23" long                        |
| Medium    | 10" long   | 16" long                             | 23" long                        |
| Large     | 12" long   | 18" long                             | 23" long                        |

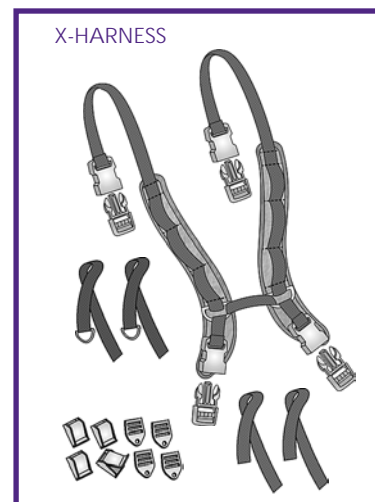

CUSTOM SEATING

BODY POSITIONING

WHEELCHAIR TRAYS


ASSISTIVE TECHNOLOGY

PATIENT TRANSFER

# HARNESSES

## PARTS DESCRIPTION

- ◆ All parts shown here are not included with every harness
- ◆ Parts included vary depending on which harness you have

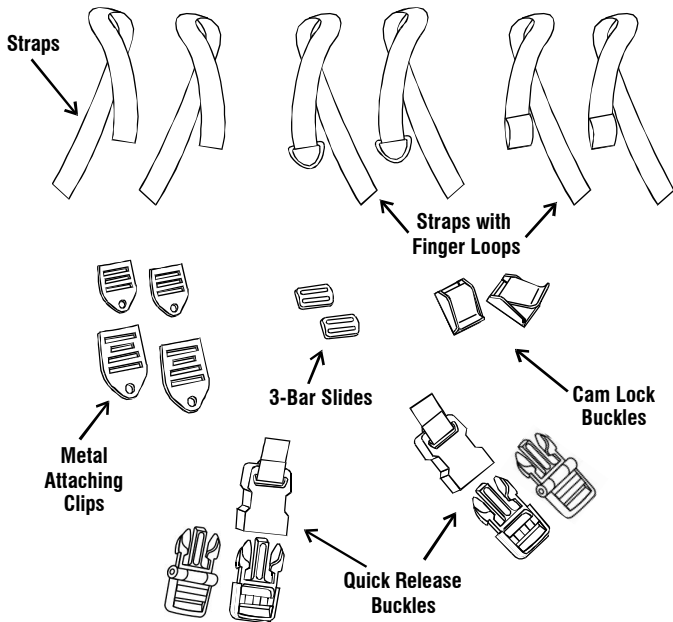


This harness is a **Kit** that you assemble as needed for your particular application. This sheet illustrates **examples** of the many ways to use the various straps, clips, and buckles.

## HOW TO ATTACH A QUICK RELEASE BUCKLE TO A STRAP

- ◆ Pull **Strap** for desired tight fit.



When threading a **Strap**, the **Buckle** piece must have the correct side up. The part the **Strap** goes around has a flat side that belongs on top, and a triangular side that belongs on the bottom.

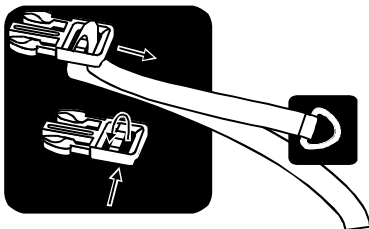


Thread the **Strap** as shown. If the **Strap** has a **Finger Loop**, the **Finger Loop** should be on the top.


## HOW TO ATTACH A STRAP TO THE CHAIR

- ◆ Secure **Cam Lock Buckle** and/or **Metal Attaching Clip** to chair, then depending on the width of the strap, thread strap as shown.


Cam Lock Buckle

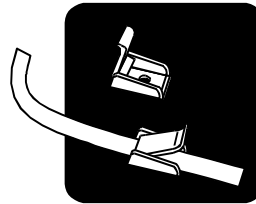


Metal Attaching Clip & 1" Strap

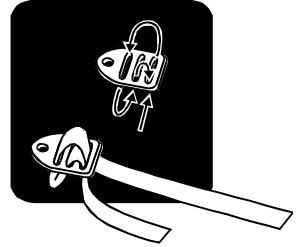


Metal Attaching Clip & 1.5" Strap

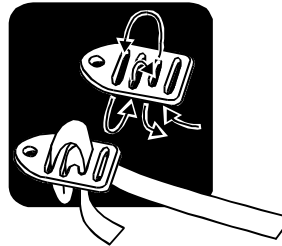

Metal Attaching Clip & 2" Strap

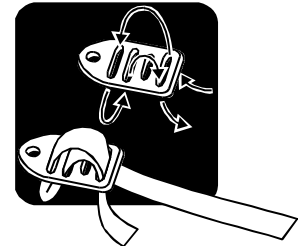

- ◆ Thread the strap through a **3-Bar Slide** as shown, looping strap around the wheelchair tube, and pull tight to hold harness in desired position.

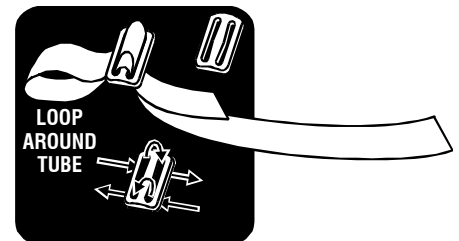

## REMOVING USER FROM CHAIR

- ◆ Use **Quick Release Buckles** so you don't have to refit all the adjustments the next time user gets into chair.

## IMPORTANT NOTICE

- ◆ **THIS IS NOT A SAFETY DEVICE. IT IS TO BE USED ONLY FOR POSITIONING. IT MUST BE USED IN CONJUNCTION WITH A METAL SEAT BELT!**